


MIC Test Strip

Antimikrobiyal Duyarlılık Testi

Minimum İnhibisyon Konsantrasyonunun belirlenmesine yönelik kantitatif test yöntemi (M.I.C.)


MIC Test Stripi antimikrobiyal ajanların mikroorganizmalara karşı minimum inhibisyon konsantrasyonunun (MIC) ve direnç mekanizmalarının belirlenmesine yönelik kantitatif bir test yöntemidir. **MIC Test Stripleri** patentli poröz yüzeyi üzerine antimikrobiyal ajan emdirilmiş striplerdir. Antimikrobiyal ajanın konsantrasyon aralığı geleneksel MIC metodundaki çift kat artan 15 farklı konsantrasyona karşılık gelmektedir.

Stribin bir yüzünde $\mu\text{g/mL}$ biriminde bir MIC skalası ve antimikrobiyal ajanı tanımlayan kısaltma yer almaktadır. ESBL, MBL, GRD, AmpC ve KPC grubu mikroorganizmalar için iki farklı antimikrobiyal ajanın kullanıldığı çift yönlü stripler mevcuttur. Oldukça geniş bir antimikrobiyal ajan çeşitliliğine sahip olan **MIC Test Stripleri** 10, 30 ve 100 testlik ambalajlarda sunulmaktadır.

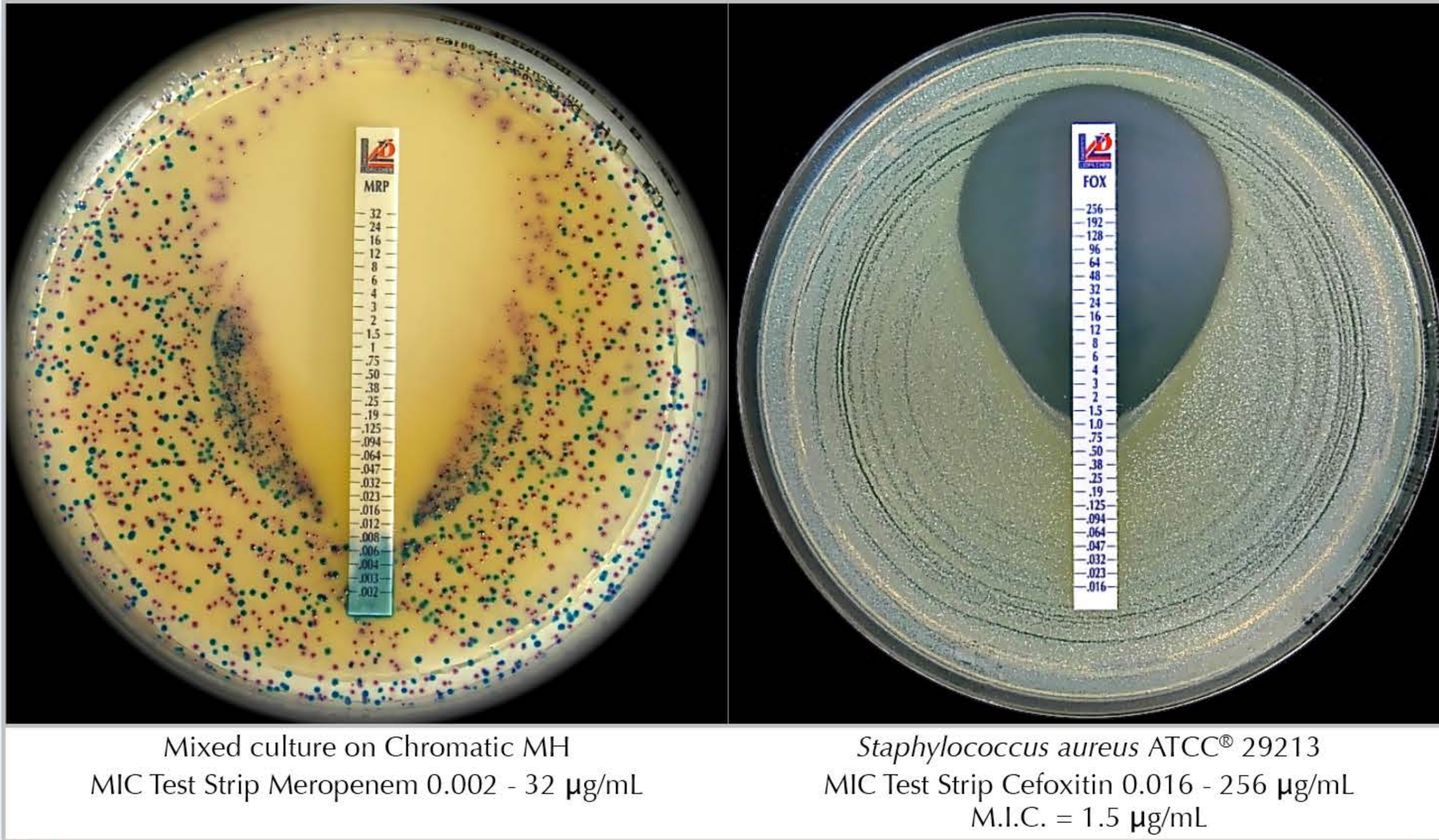


Metot prensibi

MIC Test Stribi inoküle edilmiş agar yüzeyine yerleştirildiği zaman antimikrobiyal ajan hızla ve strip boyunca artan konsantrasyonda agar matriksine difüze olur. 18 saat veya daha uzun inkübasyon sonrasında strip boyunca simetrik elips şeklinde bir inhibisyon zonu şekillenir. MIC değeri inhibisyon zonunun **MIC Test Stribi** ile keştiği noktada strip üzerindeki skaladan $\mu\text{g/mL}$ olarak okunabilir.

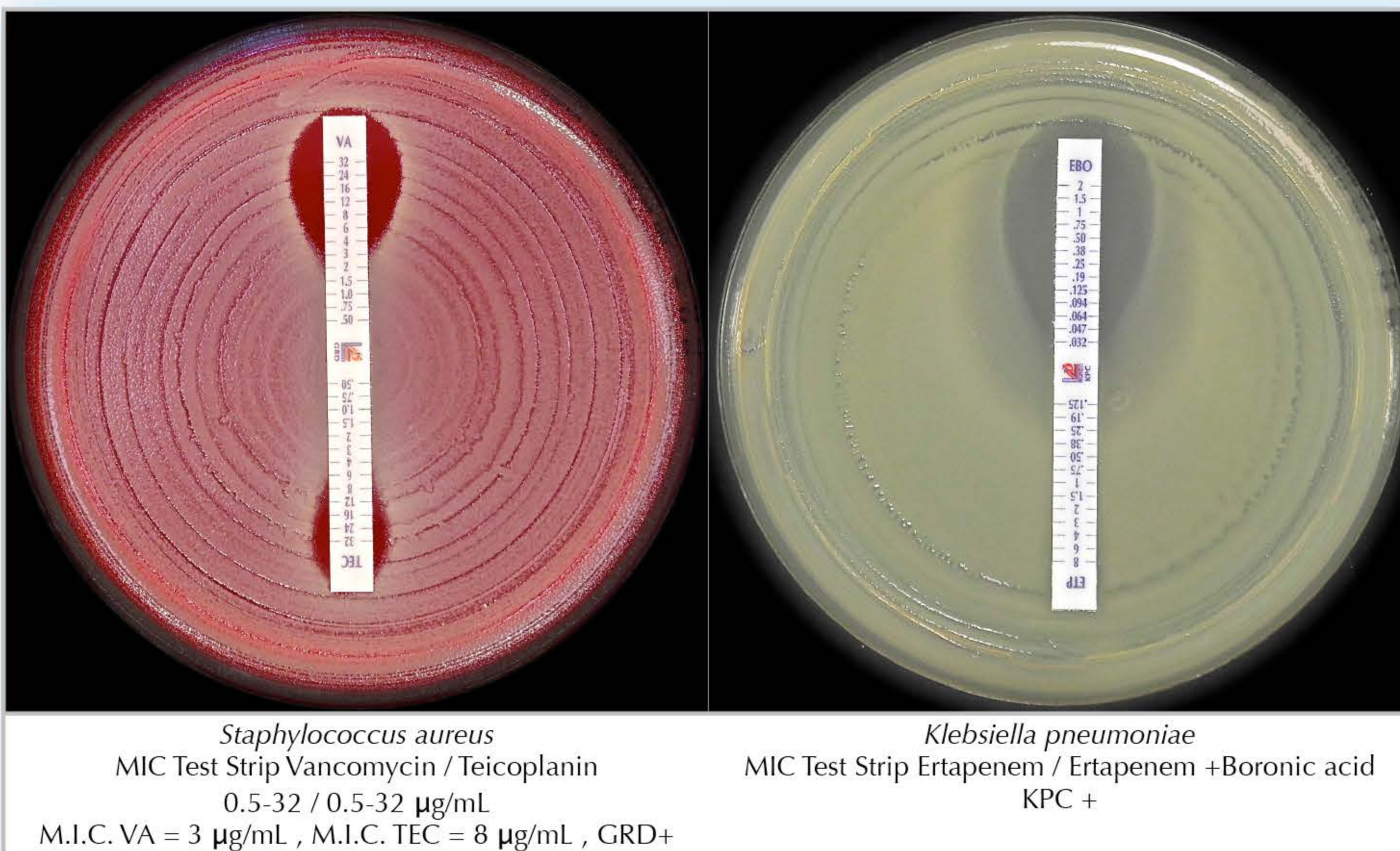

REFERANSLAR

- CLSI M100-S24 , 2014. Performance Standards for Antimicrobial Susceptibility Testing.
- CLSI M7-A9, 2012. Methods for Dilution Antimicrobial Susceptibility Tests for Bacteria that Grow Aerobically.
- CLSI M11-A8 Methods for Antimicrobial Susceptibility Testing of Anaerobic Bacteria.
- CLSI M11-S1 Performance Standards for Antimicrobial Susceptibility Testing of Anaerobic Bacteria.
- CLSI M27-A3. Reference Method for Broth Dilution Antifungal Susceptibility Testing of Yeasts; Approved Standard - Third Edition.
- CLSI M27-S4. Reference Method for Broth Dilution Antifungal Susceptibility Testing of Yeasts; Fourth Informational supplement.
- EUCAST. Breakpoint tables for interpretation of MICs and zone diameters Version 4.0 , January 2014.

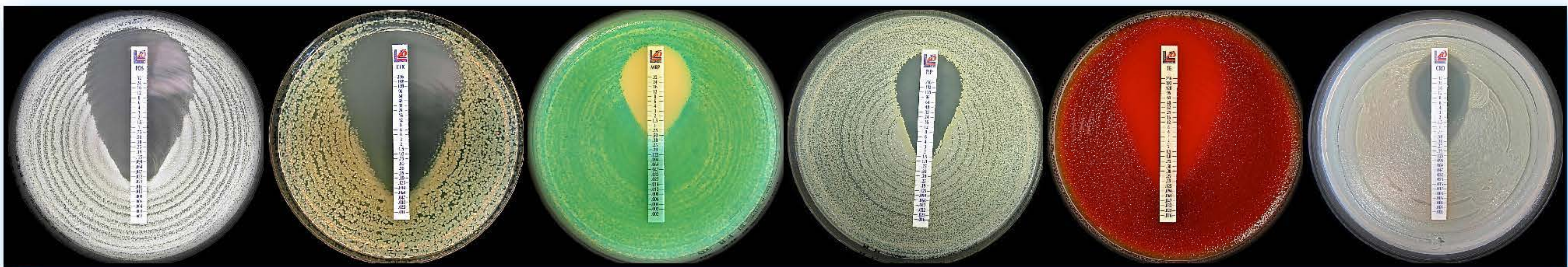

MIC Test Strip for ANTIMICROBIAL susceptibility testing					
Description		µg/mL	Code	Packaging	Ref.
MIC Test Strip	AMIKACIN	0.016 - 256	AK	10 strips	920181
				30 strips	92018
				100 strips	920180
MIC Test Strip	AMOXICILLIN	0.016 - 256	AML	10 strips	920211
				30 strips	92021
				100 strips	920210
MIC Test Strip	AMOXICILLIN*-CLAVULANIC ACID (2/1) <i>CLSI recommended</i>	0.016 - 256*	AUG	10 strips	920241
				30 strips	92024
				100 strips	920240
MIC Test Strip	AMOXICILLIN*-CLAVULANIC ACID (2 µg/mL) <i>EUCAST recommended</i>	0.016 - 256*	AMC	10 strips	921801
				30 strips	92180
				100 strips	921800
MIC Test Strip	AMPICILLIN	0.016 - 256	AMP	10 strips	920031
				30 strips	92003
				100 strips	920030
MIC Test Strip	AMPICILLIN*-SULBACTAM (2/1) <i>CLSI recommended</i>	0.016 - 256*	AMS	10 strips	920271
				30 strips	92027
				100 strips	920270
MIC Test Strip	AMPICILLIN*-SULBACTAM (4 µg/mL) <i>EUCAST recommended</i>	0.016 - 256*	SAM	10 strips	921811
				30 strips	92181
				100 strips	921810
MIC Test Strip	AZITHROMYCIN	0.016 - 256	AZM	10 strips	920301
				30 strips	92030
				100 strips	920300
MIC Test Strip	AZTREONAM	0.016 - 256	ATM	10 strips	920331
				30 strips	92033
				100 strips	920330
MIC Test Strip	AZTREONAM	0.064 - 1024	ATM	10 strips	921731
				30 strips	92173
				100 strips	921730
MIC Test Strip	BACITRACIN	0.016 - 256	BA	10 strips	920191
				30 strips	92019
				100 strips	920190
MIC Test Strip	CEFACLOR	0.016 - 256	CEC	10 strips	920361
				30 strips	92036
				100 strips	920360
MIC Test Strip	CEFEPIME	0.016 - 256	FEP	10 strips	921261
				30 strips	92126
				100 strips	921260
MIC Test Strip	CEFIXIME	0.016 - 256	CFM	10 strips	920601
				30 strips	92060
				100 strips	920600
MIC Test Strip	CEFOPERAZONE*-SULBACTAM (2/1)	0.016 - 256*	CPS	10 strips	920231
				30 strips	92023
				100 strips	920230
MIC Test Strip	CEFOTAXIME	0.002 - 32	CTX	10 strips	920071
				30 strips	92007
				100 strips	920070
MIC Test Strip	CEFOTAXIME	0.016 - 256	CTX	10 strips	920061
				30 strips	92006
				100 strips	920060
MIC Test Strip	CEFOTETAN	0.016 - 256	CTT	10 strips	920201
				30 strips	92020
				100 strips	920200
MIC Test Strip	CEFOXITIN	0.016 - 256	FOX	10 strips	920661
				30 strips	92066
				100 strips	920660
MIC Test Strip	CEFPIROME	0.016 - 256	CR	10 strips	920081
				30 strips	92008
				100 strips	920080
MIC Test Strip	CEFPODOXIME	0.016 - 256	PX	10 strips	920051
				30 strips	92005
				100 strips	920050
MIC Test Strip	CEFTAROLINE	0.016 - 256	CPT	10 strips	920491
				30 strips	92049
				100 strips	920490
MIC Test Strip	CEFTAZIDIME	0.016 - 256	CAZ	10 strips	921381
				30 strips	92138
				100 strips	921380
MIC Test Strip	CEFTIBUTEN	0.002 - 32	CTB	10 strips	920581
				30 strips	92058
				100 strips	920580
MIC Test Strip	CEFTIZOXIME	0.002 - 32	CZX	10 strips	920161
				30 strips	92016
				100 strips	920160
MIC Test Strip	CEFTRIAXONE	0.016 - 256	CRO	10 strips	920421
				30 strips	92042
				100 strips	920420

MIC Test Strip for ANTIMICROBIAL susceptibility testing

Description		µg/mL	Code	Packaging	Ref.
MIC Test Strip	CEFTRIAXONE	0.002 - 32	CRO	10 strips	920431
				30 strips	92043
				100 strips	920430
MIC Test Strip	CEFUROXIME	0.016 - 256	CXM	10 strips	921291
				30 strips	92129
				100 strips	921290
MIC Test Strip	CEPHALOTHIN	0.016 - 256	KF	10 strips	920391
				30 strips	92039
				100 strips	920390
MIC Test Strip	CHLORAMPHENICOL	0.016 - 256	C	10 strips	920751
				30 strips	92075
				100 strips	920750
MIC Test Strip	CIPROFLOXACIN	0.002 - 32	CIP	10 strips	920451
				30 strips	92045
				100 strips	920450
MIC Test Strip	CLARITHROMYCIN	0.016 - 256	CLR	10 strips	920481
				30 strips	92048
				100 strips	920480
MIC Test Strip	CLINDAMYCIN	0.016 - 256	CD	10 strips	920721
				30 strips	92072
				100 strips	920720
MIC Test Strip	CLOXACILLIN	0.016 - 256	CX	10 strips	920441
				30 strips	92044
				100 strips	920440
MIC Test Strip	COLISTIN	0.016 - 256	CS	10 strips	921411
				30 strips	92141
				100 strips	921410
MIC Test Strip	COLISTIN	0.064 - 1024	CS	10 strips	921421
				30 strips	92142
				100 strips	921420
MIC Test Strip	DAPTOMYCIN (Includes Ca ²⁺)	0.016 - 256	DAP	10 strips	921451
				30 strips	92145
				100 strips	921450
MIC Test Strip	DORIPENEM	0.002 - 32	DOR	10 strips	920401
				30 strips	92040
				100 strips	920400
MIC Test Strip	DOXYCYCLINE	0.016 - 256	DXT	10 strips	921561
				30 strips	92156
				100 strips	921560
MIC Test Strip	ENROFLOXACIN	0.002 - 32	ENR	10 strips	920131
				30 strips	92013
				100 strips	920130
MIC Test Strip	ERTAPENEM	0.002 - 32	ETP	10 strips	921571
				30 strips	92157
				100 strips	921570
MIC Test Strip	ERYTHROMYCIN	0.016 - 256	E	10 strips	920511
				30 strips	92051
				100 strips	920510
MIC Test Strip	FOSFOMYCIN (Includes Glucose-6-Phosphate)	0.016 - 256	FOS	10 strips	920781
				30 strips	92078
				100 strips	920780
MIC Test Strip	FOSFOMYCIN (Includes Glucose-6-Phosphate)	0.064 - 1024	FOS	10 strips	920791
				30 strips	92079
				100 strips	920790
MIC Test Strip	FUSIDIC ACID	0.016 - 256	FU	10 strips	920021
				30 strips	92002
				100 strips	920020
MIC Test Strip	GATIFLOXACIN	0.002 - 32	GAT	10 strips	920111
				30 strips	92011
				100 strips	920110
MIC Test Strip	GEMIFLOXACIN	0.002 - 32	GEM	10 strips	920351
				30 strips	92035
				100 strips	920350
MIC Test Strip	GENTAMICIN	0.016 - 256	CN	10 strips	920091
				30 strips	92009
				100 strips	920090
MIC Test Strip	GENTAMICIN	0.064 - 1024	CN	10 strips	920101
				30 strips	92010
				100 strips	920100
MIC Test Strip	IMIPENEM	0.002 - 32	IMI	10 strips	920541
				30 strips	92054
				100 strips	920540
MIC Test Strip	KANAMYCIN	0.016 - 256	K	10 strips	920341
				30 strips	92034
				100 strips	920340
MIC Test Strip	LEVOFLOXACIN	0.002 - 32	LEV	10 strips	920811
				30 strips	92081
				100 strips	920810
MIC Test Strip	LINEZOLID	0.016 - 256	LNZ	10 strips	921351
				30 strips	92135
				100 strips	921350
MIC Test Strip	MECILLINAM	0.016 - 256	MEC	10 strips	920171
				30 strips	92017
				100 strips	920170
MIC Test Strip	MEROPENEM	0.002 - 32	MRP	10 strips	920841
				30 strips	92084
				100 strips	920840
MIC Test Strip	METRONIDAZOLE	0.016 - 256	MTZ	10 strips	920871
				30 strips	92087
				100 strips	920870

MIC Test Strip for ANTIMICROBIAL susceptibility testing

Description		µg/mL	Code	Packaging	Ref.
MIC Test Strip	MOXIFLOXACIN	0.002 - 32	MXF	10 strips	920901
				30 strips	92090
				100 strips	920900
MIC Test Strip	MUPIROCIN	0.064 - 1024	MUP	10 strips	920381
				30 strips	92038
				100 strips	920380
MIC Test Strip	MINOCYCLINE	0.016 - 256	MN	10 strips	920321
				30 strips	92032
				100 strips	920320
MIC Test Strip	NALIDIXIC ACID	0.016 - 256	NA	10 strips	921321
				30 strips	92132
				100 strips	921320
MIC Test Strip	NETILMICIN	0.016 - 256	NET	10 strips	920931
				30 strips	92093
				100 strips	920930
MIC Test Strip	NORFLOXACIN	0.016 - 256	NOR	10 strips	920961
				30 strips	92096
				100 strips	920960
MIC Test Strip	NITROFURANTOIN	0.032 - 512	F	10 strips	920221
				30 strips	92022
				100 strips	920220
MIC Test Strip	OFLOXACIN	0.002 - 32	OFX	10 strips	920991
				30 strips	92099
				100 strips	920990
MIC Test Strip	OXACILLIN	0.016 - 256	OX	10 strips	920151
				30 strips	92015
				100 strips	920150
MIC Test Strip	PENICILLIN G	0.002 - 32	P	10 strips	921031
				30 strips	92103
				100 strips	921030
MIC Test Strip	PENICILLIN G	0.016 - 256	P	10 strips	921021
				30 strips	92102
				100 strips	921020
MIC Test Strip	PIPERACILLIN	0.016 - 256	PIP	10 strips	921051
				30 strips	92105
				100 strips	921050
MIC Test Strip	PIPERACILLIN*-TAZOBACTAM (4 µg/mL)	0.016 - 256*	TZP	10 strips	921081
				30 strips	92108
				100 strips	921080
MIC Test Strip	POLYMYXIN B	0.064 - 1024	PB	10 strips	920041
				30 strips	92004
				100 strips	920040
MIC Test Strip	QUINUPRISTIN-DALFOPRISTIN	0.002 - 32	QDA	10 strips	920261
				30 strips	92026
				100 strips	920260
MIC Test Strip	RIFAMPICIN	0.002 - 32	RD	10 strips	920011
				30 strips	92001
				100 strips	920010
MIC Test Strip	RIFAMPICIN	0.016 - 256	RD	10 strips	920251
				30 strips	92025
				100 strips	920250
MIC Test Strip	SPECTINOMYCIN	0.064 - 1024	SPC	10 strips	920141
				30 strips	92014
				100 strips	920140
MIC Test Strip	SPIRAMYCIN	0.002 - 32	SP	10 strips	920461
				30 strips	92046
				100 strips	920460
MIC Test Strip	STREPTOMYCIN	0.064 - 1024	S	10 strips	921111
				30 strips	92111
				100 strips	921110
MIC Test Strip	SULBACTAM	0.016 - 256	SUL	10 strips	920281
				30 strips	92028
				100 strips	920280
MIC Test Strip	SULFAMETHOXAZOLE	0.064 - 1024	SMX	10 strips	920311
				30 strips	92031
				100 strips	920310
MIC Test Strip	TEICOPLANIN	0.016 - 256	TEC	10 strips	920121
				30 strips	92012
				100 strips	920120
MIC Test Strip	TELAVANCIN	0.002 - 32	TLV	10 strips	920521
				30 strips	92052
				100 strips	920520
MIC Test Strip	TELAVANCIN	0.016 - 256	TLV	10 strips	920531
				30 strips	92053
				100 strips	920530
MIC Test Strip	TEMOCILLIN	0.064 - 1024	TMO	10 strips	920291
				30 strips	92029
				100 strips	920290
MIC Test Strip	TETRACYCLINE	0.016 - 256	TE	10 strips	921141
				30 strips	92114
				100 strips	921140
MIC Test Strip	TIAMULIN	0.002 - 32	TIA	10 strips	922001
				30 strips	92200
				100 strips	922000
MIC Test Strip	TICARCILLIN*-CLAVULANIC ACID (2 µg/mL)	0.016 - 256*	TTC	10 strips	921171
				30 strips	92117
				100 strips	921170
MIC Test Strip	TIGECYCLINE	0.016 - 256	TGC	10 strips	921441
				30 strips	92144
				100 strips	921440

MIC Test Strip for **ANTIMICROBIAL** susceptibility testing

Description		µg/mL	Code	Packaging	Ref.
MIC Test Strip	TILMICOSIN	0.002 - 32	TIL	10 strips	922011
				30 strips	92201
				100 strips	922010
MIC Test Strip	TOBRAMYCIN	0.016 - 256	TOB	10 strips	921211
				30 strips	92121
				100 strips	921210
MIC Test Strip	TOBRAMYCIN	0.064 - 1024	TOB	10 strips	921201
				30 strips	92120
				100 strips	921200
MIC Test Strip	TRIMETHOPRIM	0.002 - 32	TM	10 strips	920371
				30 strips	92037
				100 strips	920370
MIC Test Strip	TRIMETHOPRIM*-SULFAMETHOXAZOLE (1/19)	0.002 - 32*	SXT	10 strips	921231
				30 strips	92123
				100 strips	921230
MIC Test Strip	VANCOMYCIN	0.016 - 256	VA	10 strips	920571
				30 strips	92057
				100 strips	920570

MIC Test Strip for **ANTIFUNGAL** susceptibility testing

Description		µg/mL	Code	Packaging	Ref.
MIC Test Strip	AMPHOTERICIN B	0.002 - 32	AMB	10 strips	921531
				30 strips	92153
				100 strips	921530
MIC Test Strip	ANIDULAFUNGIN	0.002 - 32	AND	10 strips	921551
				30 strips	92155
				100 strips	921550
MIC Test Strip	CASPOFUNGIN	0.002 - 32	CAS	10 strips	921541
				30 strips	92154
				100 strips	921540
MIC Test Strip	FLUCONAZOLE	0.016 - 256	FLU	10 strips	921471
				30 strips	92147
				100 strips	921470
MIC Test Strip	FLUCYTOSINE	0.002 - 32	FC	10 strips	921491
				30 strips	92149
				100 strips	921490
MIC Test Strip	ITRACONAZOLE	0.002 - 32	ITC	10 strips	921481
				30 strips	92148
				100 strips	921480
MIC Test Strip	KETOCONAZOLE	0.002 - 32	KE	10 strips	921511
				30 strips	92151
				100 strips	921510
MIC Test Strip	MICA FUNGIN	0.002 - 32	MYC	10 strips	921821
				30 strips	92182
				100 strips	921820
MIC Test Strip	POSACONAZOLE	0.002 - 32	POS	10 strips	921521
				30 strips	92152
				100 strips	921520
MIC Test Strip	VORICONAZOLE	0.002 - 32	VO	10 strips	921501
				30 strips	92150
				100 strips	921500

MIC Test Strip for **ANTIMYCOBACTERIAL** susceptibility testing

Description		µg/mL	Code	Packaging	Ref.
MIC Test Strip	ETHAMBUTOL	0.016 - 256	EB	10 strips	921701
				30 strips	92170
				100 strips	921700
MIC Test Strip	ETHIONAMIDE	0.016 - 256	ET	10 strips	921721
				30 strips	92172
				100 strips	921720
MIC Test Strip	ISONIAZIDE	0.016 - 256	IZ	10 strips	921711
				30 strips	92171
				100 strips	921710

MIC Test Strip **ESBL** for confirmation of Extended Spectrum Beta-Lactamase

Description		µg/mL	Code	Packaging	Ref.
MIC Test Strip	CEFEPIME/CEFEPIME+CLAVULANIC ACID (4 µg/mL)	0.25-16 / 0.064-4	FEP/FEL	10 strips	921611
				30 strips	92161
				100 strips	921610
MIC Test Strip	CEFOTAXIME/CEFOTAXIME+CLAVULANIC ACID (4 µg/mL)	0.25-16 / 0.016-1	CTX/CTL	10 strips	921601
				30 strips	92160
				100 strips	921600
MIC Test Strip	CEFTAZIDIME/CEFTAZIDIME+CLAVULANIC ACID (4 µg/mL)	0.5-32 / 0.064-4	CAZ/CAL	10 strips	921591
				30 strips	92159
				100 strips	921590

MIC Test Strip for **KPC** detection

Description		µg/mL	Code	Packaging	Ref.
MIC Test Strip	MEROPENEM / MEROPENEM + BORONIC ACID	0.125-8 / 0.032-2	MRP/MBO	10 strips	921671
				30 strips	92167
				100 strips	921670
MIC Test Strip	ERTAPENEM / ERTAPENEM + BORONIC ACID	0.125-8 / 0.032-2	ETP/EBO	10 strips	921681
				30 strips	92168
				100 strips	921680

MIC Test Strip **MBL** for detection of Metallo Beta-Lactamase

Description		µg/mL	Code	Packaging	Ref.
MIC Test Strip	IMIPENEM / IMIPENEM + EDTA	4-256 / 1-64	IMI/IMD	10 strips	921621
				30 strips	92162
				100 strips	921620
MIC Test Strip	IMIPENEM / IMIPENEM + EDTA	0.125-8 / 0.032-2	IMI/IMD	10 strips	921661
				30 strips	92166
				100 strips	921660
MIC Test Strip	MEROPENEM / MEROPENEM + EDTA	0.125-8 / 0.032-2	MRP/MRD	10 strips	921651
				30 strips	92165
				100 strips	921650



MIC Test Strip **GRD** for detection of Glycopeptide Resistance

Description		µg/mL	Code	Packaging	Ref.
MIC Test Strip	VANCOMYCIN / TEICOPLANIN	0.5-32 / 0.5-32	VA/TEC	10 strips	921631
				30 strips	92163
				100 strips	921630


MIC Test Strip for **AmpC** detection

Description		µg/mL	Code	Packaging	Ref.
MIC Test Strip	CEFOTETAN / CEFOTETAN+CLOXACILLIN	0.5-32 / 0.5-32	CTT/CXT	10 strips	921641
				30 strips	92164
				100 strips	921640
MIC Test Strip	ERTAPENEM / ERTAPENEM + CLOXACILLIN	0.125-8 / 0.032-2	ETP/ECX	10 strips	921691
				30 strips	92169
				100 strips	921690

MTS Synergy Application System (patent pending)





Description		Packaging	Ref.
MTS Synergy Applicator Platform		1 unit	96860
MTS Synergy Delivery Tool		10 disposable applicators	96870

Accessories

Description		Packaging	Ref.
Automatic Rotation Plater 90 mm plates 14-110 rpm		1 unit	96795
Automatic Rotation Plater PRO (14-210 rpm)		1 unit	96796
Turntable extra wide (140 mm plates)		1 unit	96797
Wireless Foot Pedal		1 unit	96801
Steel Foot Pedal		1 unit	96798
Plastic Foot Pedal		1 unit	96799
STORAGE TUBE WITH SILICA GEL		10 tubes	96719
0.5 McFarland		1 tube	80400
1 McFarland		1 tube	80401
2 McFarland		1 tube	80402

Ready to use culture media

DESCRIPTION	FORMAT	PACKAGING	REF.
MUELLER HINTON AGAR II for antimicrobial susceptibility testing of non-fastidious organisms.	90 mm	20 plates	10031
	140 mm	10 plates	10231
MUELLER HINTON AGAR II + SHEEP BLOOD 5% for antimicrobial susceptibility testing of fastidious organisms.	90 mm	20 plates	10131
	140 mm	10 plates	11231
MUELLER HINTON FASTIDIOUS AGAR (Horse blood 5% + 20 mg/L β-NAD) EUCAST recommended: for antimicrobial susceptibility testing of <i>Streptococcus</i> spp., <i>Haemophilus</i> spp., and some other fastidious organisms.	90 mm	20 plates	10132
	140 mm	10 plates	11132
MUELLER HINTON AGAR II + 2% NaCl for antimicrobial susceptibility testing of staphylococci with Oxacillin.	90 mm	20 plates	11206
CHROMATIC MH Chromogenic medium for the preliminary identification and susceptibility testing of bacteria directly from clinical and environmental specimens.	90 mm	20 plates	11618
BRUCELLA BLOOD AGAR with HEMIN AND VITAMIN K1 for antimicrobial susceptibility testing of anaerobic bacteria.	90 mm	20 plates	10245
SCHADLER K AGAR (SHEEP BLOOD 5%) for antimicrobial susceptibility testing of anaerobic bacteria.	90 mm	20 plates	11065
HAEMOPHILUS TEST AGAR for <i>Haemophilus</i> spp. susceptibility testing.	90 mm	20 plates	10080
MIDDLEBROOK 7H11 AGAR for antimicrobial susceptibility testing of mycobacteria.	90 mm	20 plates	10416
R.P.M.I. AGAR (2% glucose + MOPS) for antifungal susceptibility testing.	90 mm	20 plates	11509
	140 mm	10 plates	10233

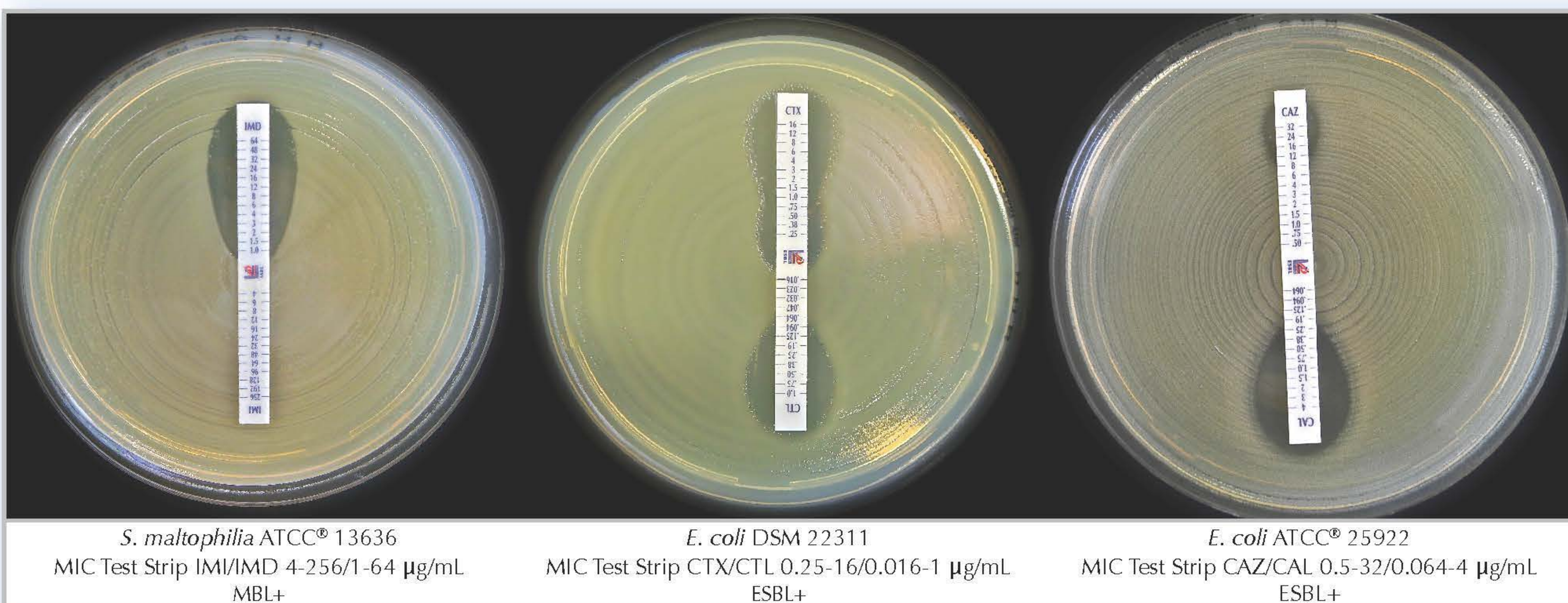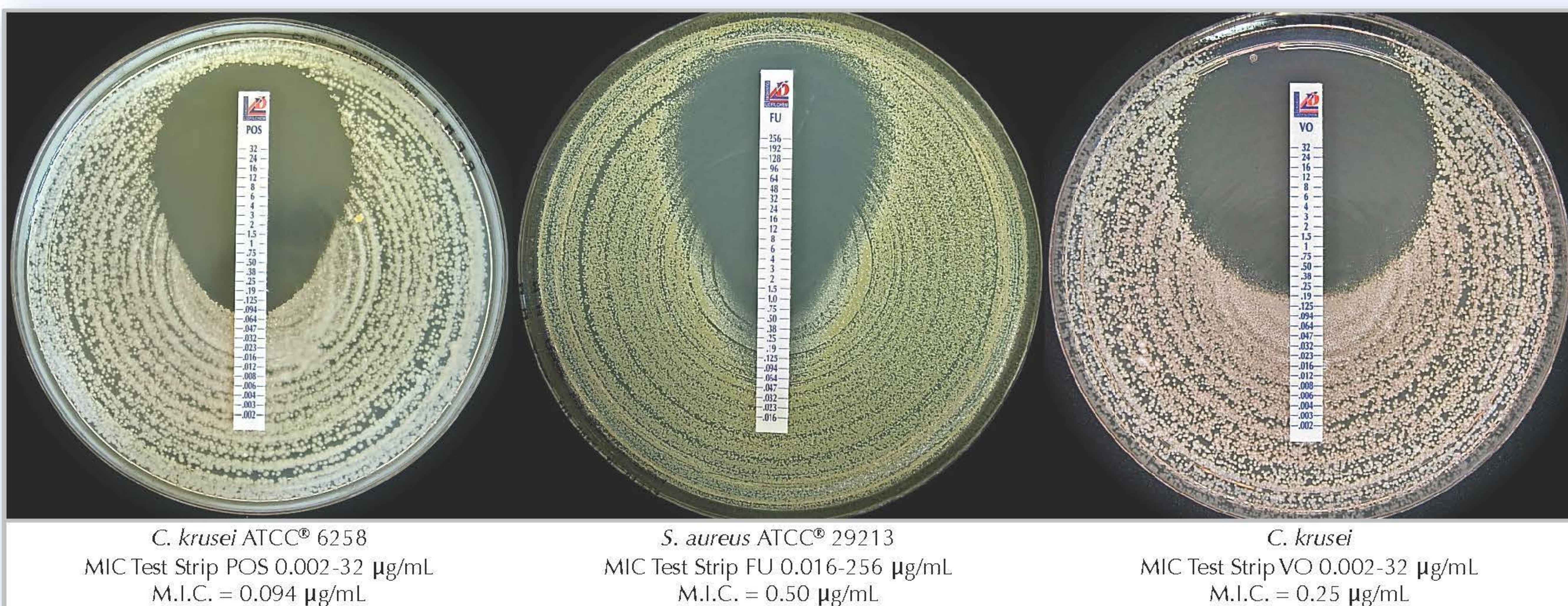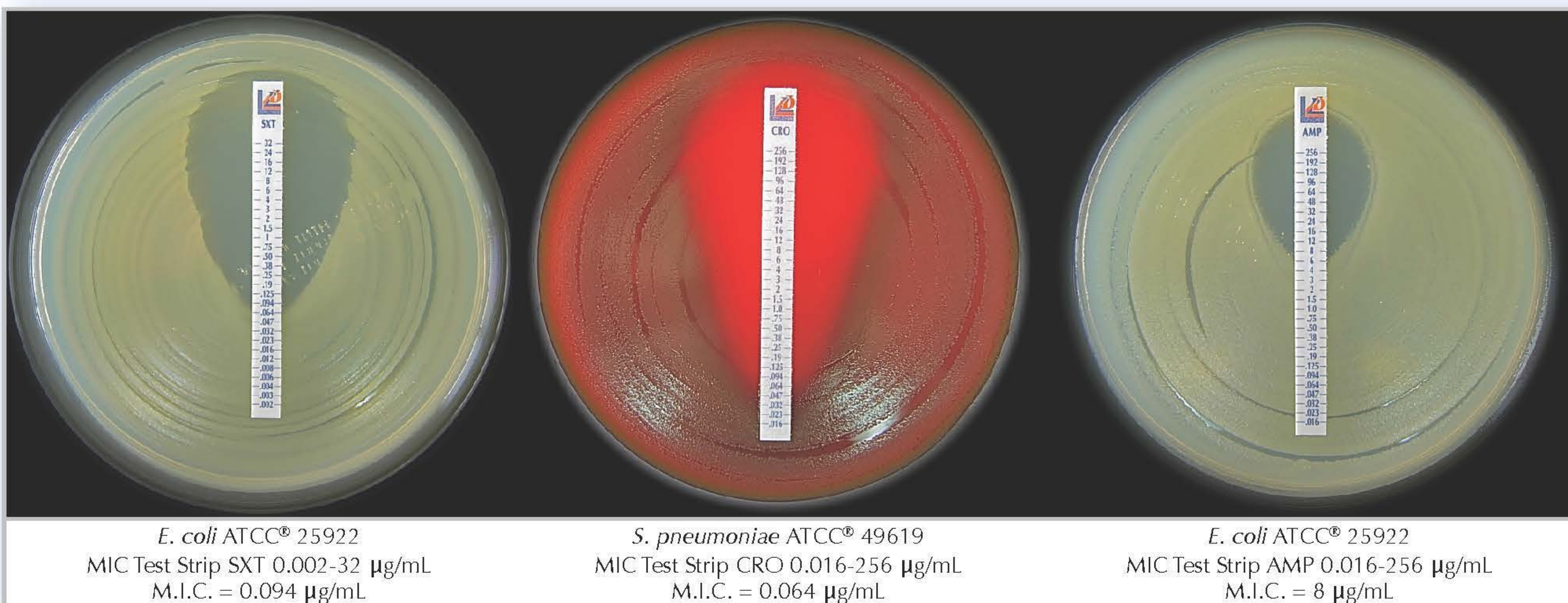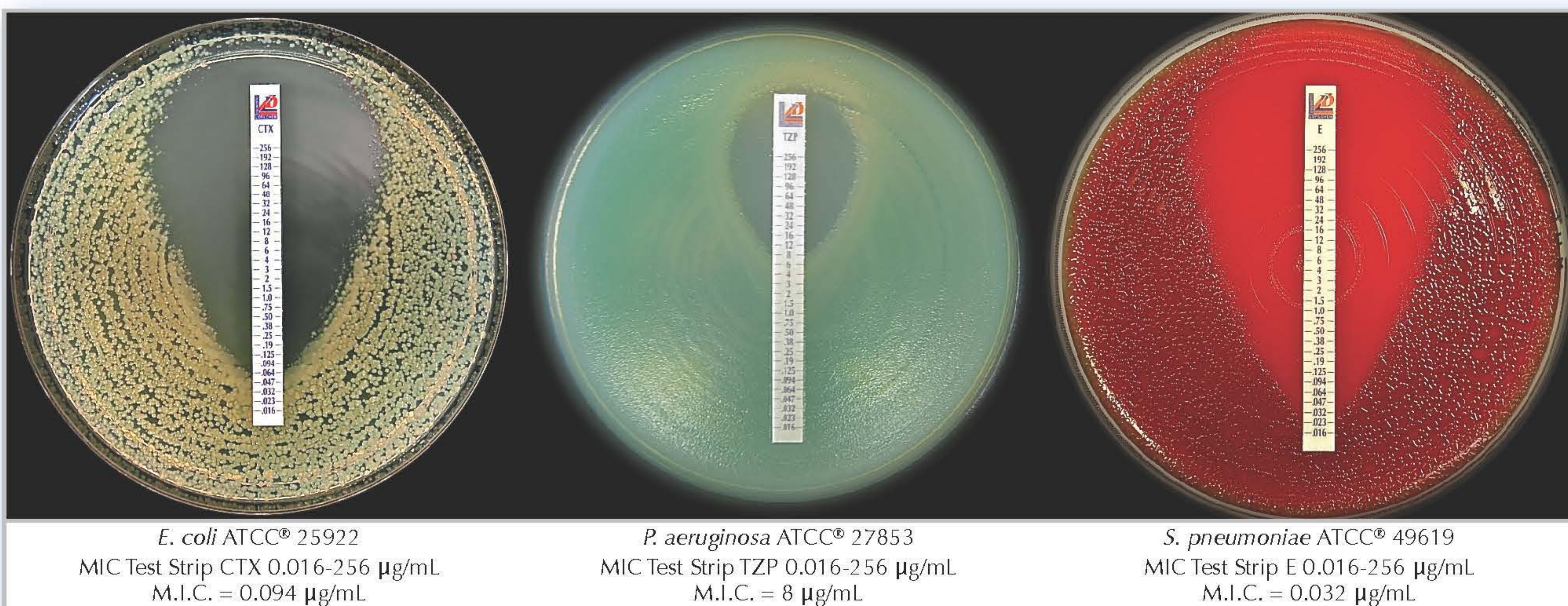

Contact your local dealer or Liofilchem headquarter for **MIC Test Strip** availability.



Algeria
Sarl Dimed
Tel +213 (0)21 75 53 80
Fax +213 (0)21 75 53 91
www.dimed-dz.com



Denmark
Biotech-IgG A/S
Tel +45 3538 0500
Fax +45 2538 7322
info@biotech-igg.com



Italy
Liofilchem
Tel +39 0858930745
Fax +39 0858930330
www.liofilchem.it



Saudi Arabia
Gamsco
Tel +966-3-8394085
Fax +966-3-8394084
drmalousef@yahoo.com



Argentina
MedicaTec S.R.L.
Tel +54 11 4554.4600
Fax +54 11 4555.0416
www.medica-tec.com



Egypt
Media Medix Trade
Tel +20 22598417
Fax +20 24519256
mediamo_co@yahoo.com



Japan
Alere Medical Co., Ltd.
Tel +81353267393
Fax +81353267177
www.alere.co.jp



Serbia
Uni-Chem
Tel +381113282986
Fax +381113281205
www.uni-chem.rs



Australia
Integrated Sciences Pty Ltd
Tel +61 2 9417 7866
Fax +61 2 9417 5066
www.integratedsci.com.au



Estonia
Borsa Eesti OÜ
Tel +3725050555
Fax +3726556158
borsa@borsa.ee



Lithuania
Biometrija
tel +37 052481824
fax +37 052400085
www.biometrija.lt



Singapore
Premier Diagnostics Pte Ltd
Tel +65 6220 3221
Fax +65 6220 9457
pdsingapore8@gmail.com



Belarus
Multi Lab Ltd.
Tel.: +375 17 309 17 20
Fax: +375 17 299 09 02
multilab@multilab.by



Finland
Immuno Diagnostic Oy
Tel +3583615370
Fax +35836822039
www.immunodiagnostic.fi



Malaysia
Premier Diagnostics Sdn Bhd
Tel +603-8024-0680
Fax +603-8024-0780
www.premierdiagnostics.com.my



Slovakia
Bio G spol. s r.o.
Tel +421245526381
Fax +421245526382
www.biog.sk



Benelux
ELITech Benelux B.V.B.A
Tel +32 (0)92820531
Fax +32 (0)92820532
c.notte@elitechgroup.com



France
i2a
Tel +33467504800
Fax +33467170906
www.i2a.info



New Zealand
Point View Laboratories Ltd
Tel +64 9 533 3380
Fax +64 9 533 3340
info@pointviewlaboratories.com



Slovenia
Mediline d.o.o.
Tel +38618308050
Fax +38618308070
www.mediline.si



Bulgaria
Elta 90M
Tel +35929839649
Fax +35929832211
www.elta90.com



Germany
bestbion dx GmbH
Tel +49 2234 987 95 0
Fax +49 2234 987 95 29
bernd.stammel@bestbion.com



Norway
Montebello Diagnostics AS
Tel 22 14 14 90
Fax 22 14 44 90
www.montebello.no



South Africa
Davies Diagnostics (PTY Ltd)
Tel +27117777600
Fax +27117876917
www.daviesdiagnostics.co.za



Canada
Alere Canada
Tel (613) 271-1144
Fax (613) 271-1148
www.alere.ca



Greece
Varelas S.A.
Tel +302105281920
Fax +302105220926
microbiology@varelas.gr



Pakistan
Imran Scientific Co
Tel 021-32210083
Fax 021-32210630
www.imransco.com



Spain
IZASA Distribuciones Técnicas, S.A.
Tel 902203090
Fax 902223366
www.izasa.es



Chile
Medica Tec de Chile S.A.
Tel +56 (02) 4261830
fax +56 (02) 4261835
mrojas@medica-tec.cl



Hungary
Frank Diagnosztika Kft.
Tel 388-3114, 367-2430
Fax 368-5721
www.frank-diagn.hu



Paraguay
GT Scientific S.A.
Tel 595 21 200134
Fax 595 21 221339
ventas@gtscientific.com.py



Sweden
Biotech-IgG AB
Tel +46 46 12 37 40
Fax +46 46 12 37 60
ak@biotech-igg.com



Croatia
Tamiko Instruments
Tel +38514683046
Fax +38514684811
www.tamiko.hr



Indonesia
PT Suryana Microbiology
Tel 62 21 345 2783
Fax 62 21 384 7232
najib@ptsuryana.com



Portugal
Izasa Portugal Distribuições Técnicas Lda.
Tel +351 214161288
Fax +351 214176484
MIC Test Strip product range only



Thailand
Envimed Co., Ltd.
Tel 66-2-434-0391
Fax 66-2-435-5962
envimed@truemail.co.th



Cuba
CPM Cuba
Tel 2041432-2041433
Fax 2040987
microbiologia@cpm.co.cu



Iran
Pishro Imen Pajouh Co.Ltd.
Tel +98 21 44328639
Fax +98 21 44311632
pipco.ir@gmail.com



Romania
S.C. Sanimed International Impex SRL
Tel 021.420.54.94
Fax 021.420.54.93
www.sanimed.ro



Turkey
Kemiteks Kimyevi Mad. Tic. Ltd. Sti.
Tel +90 216 449 58 70
Fax +90 216 449 58 72
www.kemitekskimya.com.tr



Czech Republic
BioVendor
Tel +420549124123
Fax +420549211465
www.biovendor.cz



Ireland
Launch Diagnostics Ireland
Tel +3530719151122
Fax +3530719152277
desruddy@launchdiagnostics.ie



Russia
Agros-International, Zao
Tel +7 (812) 303-8234
Fax +7 (812) 303-8236
agros@agros-int.com



United Kingdom
Launch Diagnostics Ltd
Tel +44(0)1474874426
Fax +44(0)1474872388
enquiries@launchdiagnostics.com



Liofilchem®

Via Scozia zona ind.le, 64026 Roseto degli Abruzzi (Te) Italy

Tel. +39 0858930745

Fax +39 0858930330

www.liofilchem.net

liofilchem@liofilchem.net



ref. 6553003
Rev.14.1 / 29.05.2014